Need Help?

For more information, go to www.diabetes.ascensia.com

Your meter is preset and locked to display results in mmol/L. If your result is in mg/dL, contact Customer Service.

Imported and Distributed by: Ascensia Diabetes Care UK Ltd	Imported and Distributed by: Ascensia Diabetes Care Ireland Ltd
Ascensia House	10 Earlsfort Terrace
Albert Road	Dublin 2
Newbury, Berkshire	D02 T380
RG14 1DL UK	Ireland
Ascensia Diabetes Support	Ascensia Diabetes Support
0345 600 6030 – UK	1 800 920 111 – IE

For in vitro diagnostic use.

Ascensia, the Ascensia Diabetes Care logo, Contour, Microlet, the o Coding logo. Second-Chance, the Second-Chance sampling logo Smartcolour, and Smartlight are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG.

Apple and the Apple logo are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc. The **Bluetooth®** word mark and logos are registered trademarks owned by Bluetooth SIG, Inc., and any use of such marks by Ascensia Diabetes Care s under license

Google Play and the Google Play logo are trademarks of Google LLC. All other trademarks are properties of their respective owners. No elationship or endorsement should be inferred or implied.

EC REP Ascensia Diabetes Care Italy S.r.I. Via Varesina, 162 20156 Milano, Italy

© 2024 Ascensia Diabetes Care Holdings AG. All rights reserved

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Turn the Meter On and Off

- Press and hold the **OK** button for about 3 seconds until your meter turns On or Off.
- Inserting a test strip into the meter will also turn the meter On.
- Press OK twice when the meter is Off to turn On the white test strip port light.

Home Screen

Button

Strip Port

Test

- Sample Tip: Blood

sample pulled in here

The Home screen 10:45 12,4 has 2 options: Logbook and Settings.

- Press the button until the option you want is blinking
- · To see your past readings and Averages press the **OK** button while **Logbook** is blinking. See Logbook and View Averages.
- To change your Settings, press the OK button when Settings is blinking. See Settings.

For a table of symbols, see *Your Meter* Display on the second side.

NOTE: When a symbol is blinking, you can select it.

Fingertip Testing

II Refer to your lancing device insert for detailed instructions on preparing the lancing device.

WARNING

- otential Biohazard Always wash your hands with soar
- lancing device, or test strips. All blood glucose measuring system are considered biohazardous. Healt care professionals or persons using
 - this system on multiple patients should follow the infection control
- nust not be used on more than one person due to the risk of infection.

mall Parts

- Keep out of reach of children. This kit
 - Keep batteries away from children Many types of batteries are poisonous poison control centre.

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and after testing or handling the meter

The lancing device provided is intende

suffocation if accidentally swallowed. If swallowed, immediately contact your

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- **1.** Prepare the lancing device. Refer to your lancing device insert for detailed instructions.
- 2. Insert the grey square end of a new test strip firmly into the test strip port until the meter beeps

The meter turns On and displays the **Apply Blood** screen. The meter is now ready to test a blood drop.

- **3.** Use your lancing device to get a drop of blood from your fingertip
- 4. Stroke your hand and finger toward the puncture site to form a drop of blood.
- 5. Immediately touch the tip of the test strip to the drop of blood.

Second-Chance samplin

If the meter beeps twice and the screen displays a blinking blood drop with a plus sign, the test strip does not have enough blood.

Do not remove the strip. You have 60 seconds to apply more blood to the same test strip.

Meal Markers

When Meal Markers are On, you can mark readings as Fasting 🚫, Before Meal 🖤, or After Meal T. See Turn Meal Markers On.

smartLIGHT target range indicato

The meter displays your result with the units, time, date, and target indicator:

The test strip port light displays a colour that indicates if your reading is above, in, or below your personal Target Range.

Logbook

00:60

+6

00:60

+

- 1. To turn on your meter, press and hold the OK button for 3 seconds. The Logbook is blinking.
- 2. To view your **Logbook** entries, press the **OK** button to enter the Logbook.

10:45 12.4

- 3. To scroll through the Logbook, press the ▲ or ▼ button. To scroll guickly through entries, press and hold the \blacktriangle or \triangledown button.
- 4. To exit the **Logbook** and return to the **Home** screen at any time, press the OK button.

View Averages

- 1. To enter the Logbook from the Home screen, press the **OK** button while the Logbook III is blinking.
- 2. To view your Averages, press the ▲ button from the *first* Logbook screen.

4. To return to your readings, press the 🔻 button on the 7 d Avg

(7-day Average).

5. To exit Averages and return to the Home screen at any time, press the OK button.

7-day

Settings

You can change the settings for:

- Date: Change format and date.
- Time: Change format and time.
- Meal Markers 🕅 🖤 🎽 Turn Meal Markers On or Off.
- Bluetooth wireless technology X: Turn Bluetooth wireless functionality On or Off.
- smartLIGHT feature 👾 : Turn smartLIGHT feature On or Off. See smartLIGHT target range indicator.

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🔟 blikkar.

10:45 12.4 mælinum skaltu

- Target Ranges @ : View and change Target Ranges.
- Sound (): Turn Sound On or Off.
- **Reminder** (): Turn the Reminder feature On or Off.

NOTE: To change the meter time and date see the user guide.

To change meter settings:

- 1. To highlight the **Settings** symbol 🌞 from the **Home** screen, press the ▼ button.
- 2. When the Settings symbol is blinking, press the **OK** button to
- enter Settings.
- 10:45 12.4

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Stillingar, framhald á næstu hlið 📥

Settings (continued)

- 3. To scroll through 10:45 12.4 Settings options as shown in the image, press the ▲ or ▼ button until Ö **€**@ the symbol you
- want is blinking.
- 4. Press the **OK** button to select the blinking symbol.
- **5.** To edit a Setting, press the \blacktriangle or \checkmark button. You can turn the Setting On or Off, or change the value.
- 6. To accept the change and return to the Home screen, press the OK button.

To exit from Settings and return to Home:

0

- **1.** Press the \blacktriangle or \blacktriangledown button until the **Settinas** symbol 🌞 is blinking.
- 2. Press the OK button.

If you are in a Setting such as Date and you need to exit, press the OK button repeatedly until you get back to the Home screen.

WARNING

fore setting any **Target Ranges** in ur meter or in compatible Ascensia abetes Care software.

If Meal Markers are Off, you can set only 1 Target Range. To set Before Meal and After Meal Target Ranges, see the CONTOUR NEXT user guide.

- 1. To highlight the **Settings** symbol **‡** from the **Home** screen, press the **▼** button.
- 2. When the **Settings** symbol **‡** is blinking, press the **OK** button to enter **Settings**.
- 3. To scroll to the Target Range symbol of, press the \blacktriangle or \checkmark button. When the symbol is blinking, press the **OK** button.
- 4. To set the blinking Overall Low Target, press the \blacktriangle or \checkmark button, then press the OK button.

Ö.

3.9 -

Marker symbols are blinking, press the **OK** button.

5. To set the blinking

Target, press the

Overall High

▲ or ▼ button,

then press the

5. Press the ▲ or ▼ button until On is blinking, then press the OK button. NOTE: With Meal Markers On, you can set Before Meal and After Meal Target Ranges (see the CONTOUR NEXT user guide). You can also set a Reminder.

Turn Reminder Feature On

When Meal Markers are On, you can set a **Reminder** to test your blood glucose after you mark a reading as Before Meal. If Meal Markers are Off, see Turn Meal Markers On in this guide.

- 1. To highlight the Settings symbol 🌞 from the **Home** screen, press the **▼** button.
- 2. When the Settings symbol 🌞 is blinking, press the OK button to enter Settings.
- **3.** To scroll to the **Reminder** symbol (), press the \blacktriangle or \checkmark button.
- 4. When the Reminder symbol 🕑 is blinking, press the **OK** button.

5. Press the
or button until On is blinking, then press the OK button.

Select a Meal Marker During a Blood Glucose Test

Meal Marker	S
🕺 Fasting	Ľ
Before Meal	>

NOTE: You must do a blood glucose test before you can select a Meal Marker. You cannot select a Meal Marker in Settings.

- **1.** Test your blood glucose. See *Fingertip* Testing in this guide.
- 2. Do not remove the test strip until you select a Meal Marker

A Meal Marker blinks on the blood glucose

- result screen. 3. If the blinking Meal Marker is the one you want, press the OK button. To select a different Meal Marker, press the ▲ or ▼ button.
- 4. When the Meal Marker you want is blinking, press the **OK** button.

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á **OK**-hnappinn.

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mbols After Meal o Mark

NOTE: If you do not select a Meal Marker

within 3 minutes, the meter turns Off. Your

I. Make sure the **Reminder** () feature is On

2. Mark a blood glucose reading as a Before

Meal reading, then press the OK button.

blood glucose reading is stored in the

Logbook without a Meal Marker.

Set a Reminder

in Settings.

3. To scroll from

2 hours to

button.

0.5 hours by

half-hour intervals,

4. To set the **Reminder**, press the

press the **A** or

LO Results

12 4

Follow medical advice immediately. Contact your health care professional.

HI Results

If the meter beeps once and displays the HI screen, your blood glucose reading is over 33.3 mmol/L.	10:45	- -

1. Wash and dry your hands well. 2. Retest with a new strip.

If results are still over 33.3 mmol/L,

follow medical advice immediately.

To turn the meter Off, remove the test strip.

WARNING

- Always dispose of used test strips and lancets as medical waste or as advise by your health care professional.
- Health care professionals should follow the biohazard disposal requirements for their facility.

15

12 4

12.4

Π

Your Meter Display

	-		
Symbol	What It Means	Symbol	What It Means
Ш	Your Logbook	**	Fasting marker; used when testing after fasting (no
₩.	Meter Settings	\bigtriangledown	except water or non-caloric beverages)
-``#	smartLIGHT target range indicator (Yellow=Above Target; Green=In Target; Red=Below Target)	Ŭ	Before Meal marker; used when testing within 1 hour before a meal
Ø	A Target Range or Target Range setting	Ĭ	After Meal marker; used when testing within 2 hours after the first bite of a meal
	Blood glucose test result is above Target Range	×	No Meal marker selected
\checkmark	Blood glucose test result is in Target Range		Reminder feature
♥	Blood glucose test result is below Target Range	•)	Sound feature

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Symbol	What It Means		
*	Indicates the Bluetooth wireless setting is On; the meter can communicate with a mobile device		
Ε	Indicates a meter error		
	Batteries are low and need to be changed		
ld Avg	7-, 14-, 30-, and 90-day averages		
П	Total number of blood glucose readings used to calculate averages		
	Control solution test result		
+ 🌢	Add more blood to same test strip		

The CONTOUR DIABETES app allows you to transfer results from your meter to your compatible smartphone or tablet. See the meter user guide for instructions and features.

		-	
1	1	C	2
		E	3
-			

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